

# Renray Guide to Handset Functions for Rise and Recline Chairs

---

## Single motor handset

- 1 Enables the chair to move to a rise position
- 2 Returns the chair to a seated position, followed by the footrest being raised. On a tilt-in-space chair the seat and back will start to recline together. By keeping the button depressed the back will then recline



## Dual/3 motor max handset

- 1 Moves the footboard to a raised position
- 2 Returns the footboard to a closed position
- 3 Reclines the back
- 4 Returns the back to a upright position
- 5 Enables the chair to move to a rise position
- 6 Returns the chair to a seated position



## 3 motor, extra max and bariatric handset

- 1 Moves the footboard to a raised position.  
Returns chair to seated position when chair has been in rise position
- 2 Returns the footboard to a closed position. Keeping this button pressed will move the chair into a rise position
- 3 Reclines the back
- 4 Returns the back to a upright position



## 2 motor 4th position handset

- 1 Enables the chair to move to a rise position
- 2 Moves the footboard to a raised position followed by the seat going into a tilt position
- 3 Reclines the back
- 4 Returns the back to a upright position



## 2 motor portable handset

- 1 Moves the footboard to a raised position
- 2 Returns the footboard to a closed position
- 3 Reclines the back
- 4 Returns the back to a upright position
- 5 Raises the footboard and reclines the back simultaneously
- 6 Returns the back and footboard to a upright and closed position respectively

